---

**FR – Instructions d'installation**

---

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## **EN – Installation instructions**

---

You will find below the link and the instructions to download and install the Global D libraries. For more information, visit our website <https://www.globald.com/en/implantology/implantology-digital-solutions/cad/>

**DentalWings folder containing the download link libraries:**

Global\_D\_DW\_library\_complète\_v2.2.2+In-Kone\_WD\_v2.4.5\_Pack Link :  
<https://monextra.net/wl/?id=fUMfHA8UaKIztSL6RJU6L9C7H28CXmeY>  
password: **Gld2018-Dwos**  
Unzip Global\_D\_DW\_library\_complète\_v2.2.2+In-Kone\_WD\_v2.4.5\_Pack.zip

Extract the following files:

- Global D Titanium Premilled S3DEL.xml
- Global D v2.2.2.zip
- Global D WD V2.4.5.zip

If you cannot control the installation of a new library, contact our software vendor.

**Installation of the Dental Wings library:**

- Select the icon parameter (gear) / Materials
- If "Global D" does not exist, create the manufacturer "Global D" Clicking the + sign at the bottom right of the window DWOS,
- Install or update Global\_D\_Titanium\_premilled\_S3DEL.xml material (right-click Global D / import material)
- Select the setting icon (gear) / Implant Kits
- Delete all manufacturers starting with Global D (select the row, right click "delete the manufacturer")
- Click on the + sign at the bottom right of the DWOS window
- Select import manufacturer implant kits
- Choose the file "Global D v2.2.2.zip"
- Click on the + sign at the bottom right of the DWOS window
- Select import manufacturer implant kits
- Choose the file "Global D WD V2.4.5.zip"

---

**Added in-Kone WD librarie:**

- Select the icon parameter (gear) / Materials
- If "Global D" does not exist, create the manufacturer "Global D" Clicking the + sign at the bottom right of the window DWOS,
- Install or update Global\_D\_Titanium\_premilled\_S3DEL.xml material (right-click Global D / import material)
- Select the setting icon (gear) / Implant Kits
- Click on the + sign at the bottom right of the DWOS window
- Select import manufacturer implant kits
- Choose the file "Global D WD V2.4.5.zip"

**Selection menus:**

- "Global D V2.2" contains V2.2.2 standard library to use
- "Global D WD V2.4" additional library with only the prosthetic connection on In-Kone WD implant

**Warning**

The implant kits "In-Kone + PC4.3" and "EVL + MULTI" are only available for coDiagnostiX implant plans via the Synergy module.

---

**The CAD-CAM connections provided:**

- **In-Kone ST** on implant and indexed Ti Base In-Kone Ø3.8 Gingival heights 0.6, 2, 3 and 4 mm or Ø5.5 heights 1.5, 2.2, 3 et 4 mm,
- **In-Kone WD** on implant and indexed Ti Base In-Kone for healing screws Ø6.5, Gingival heights 1.5, 2.2, 3 et 4 mm,
- **EVL** on implant and indexed Ti Base EVL (except EVL C),
- **EVL NI** on implant and Ti Base non-indexed EVL (except Exocad – 100% panthera)
- **PC 4.3** on new conical abutments 0 ° and rights associated sockets, non-indexed (In-Kone, TwinKon)
- **PAOF** on angulated conical abutments and associated sockets, non-indexed In-Kone (In-Kone, compatible with Hexacolor Ø4.7 angulated conical abutment)
- **MULTI** on MULTI abutment and bases, implant EVLC, non-indexed (EVL, CLINIC, compatible with In-kone PC4.0 and Hexacolor conical abutment)
- **MUA** on pillars and sockets MULTI angulated, non-indexed (EVL, CLINIC,).
- **In-Kone+PC4.3 or PC4.3 implant Studio** reserved in synergy with a guided surgery planning,
- **EVL + MULTI** reserved in synergy with a guided surgery planning.

**Examples of selection:**

- For designing an abutment on titanium base, please select ...> ...**BaseTi**
- For designing an abutment on titanium base 2 mm shortened on lab (top shaft = 3 mm instead of 5 mm (standard)), please select ... > ...**BaseTi - 3mm**
- For designing a customized abutment from premilled on S3DEL assembly on lab (In-Kone example), please select **Global D V2 > Global D In-Kone > Inkone Premilled S3DEL**,
- For designing a customized Bridge or bar « Compliance » without Ti Base (PC 4.3 example), please select **Global D V2 > Global D PC4.3 > PC4.3 Compliance**.

For more information see file "**DigitalSol\_CAD-CAM\_QuickStart...**"

This library includes several heights of In-Kone bases and offers the possibility to the lab to shorten them to 3 mm (i.e. at the level of the middle groove). **Warning 3D direct connections implants or abutments are false. Only the Panthera Dental Company is authorized to machine these connections. Machining Laboratory is possible via Tibase, premilled or compliance system.**

We count on you to let us know your suggestions for correction or improvement.